

Selecting Training Scenarios

You can select a training scenario by using either the optional remote control or via the Trainer's set-up mode. To activate the setup mode, make sure the electrode pads are not connected, then remove and reinsert the battery. Alternatively, press and hold down the two blue Up/Down buttons while simultaneously pressing the green On/Off button. When the Trainer is in set-up mode, the operator display shows the currently selected scenario. To change the scenario selection, use the Up/Down buttons to select the desired scenario, then press the On/Off button to exit the setup mode. The last scenario selection is stored in the unit even if the battery is taken out.

PREPROGRAMMED SCENARIOS IN AED TRAINER 2

Scenario#	Description		ARC* Senario#	AHA** Senario#
1	Ventricular Fibrillation - with single-shock conversion	- Shockable rhythm - 1 shock - Non-shockable rhythm		
2	Ventricular Fibrillation - with multiple shocks required for conversion	- Shockable rhythm - 3 shocks - 1 minute CPR time-out - 4th shock - Non-shockable rhythm	ARC#1 ARC#3	AHA #1 AHA #5
3	Troubleshooting Skills - defibrillation pads	- Poor pad connection - Shockable rhythm - 1 shock - Non-shockable rhythm		
4	Ventricular Fibrillation - conversion with refribrillation	- Shockable rhythm - 1 shock - Non-shockable rhythm - Refibrillation - shockable rhythm - 2nd shock - Non-shockable rhythm	ARC#2 ARC#4	
5	Non-Shockable Rhythm	- Non-shockable rhythm throughout		AHA #4 AHA #6
6	Ventricular Fibrillation - with two shocks required for conversion	- Shockable rhythm - 2 shocks - Non-shockable rhythm		AHA #2 AHA #3
7	Ventricular Fibrillation - with two shocks required for conversion, followed by refribrillation	- Shockable rhythm - 2 shocks - Non-shockable rhythm - Refibrillation - shockable rhythm after 50 seconds - 3rd shock - Non-shockable rhythm		AHA #7
8	Troubleshooting Skills - defibrillation pads with two shocks required for conversion	- Poor pad connection for 15 sec. - Shockable rhythm - 2 shocks - Non-shockable rhythm		AHA #8
9	Ventricular Fibrillation	- Ventricular fibrillation throughout		
10	Troubleshooting Skills - motion, low battery, with one shock required for conversion	- Motion artefacts - Shockable rhythm - 1 shock - Non-shockable rhythm - Battery low		

* ARC - American Red Cross

** AHA - American Heart Association

Product specifications are subject to change without notice.